

schülke -t



Ready to use disinfectant wipes impregnated with an alcoholic solution.

mikroqid[®] AF jumbo wipes

Our plus:

- ready to use, impregnated disinfection wipes
- very broad biocidal activity within the shortest possible time
- dries rapidly
- 200 wipes per dispenser / refill pack
- jumbo wipe (20 x 27 cm)

Areas of application

Alcohol-based rapid disinfection of medical devices [CE 0297] in all areas with an increased risk of infection and where short contact times are required, e. g.:

- patient treatment units
- examination couches
- operating tables and nearby work surfaces
- surfaces of medical equipment

Microbiological effectiveness

- bacteria (including TB) • fungi • Hepatitis B viruses • HIV • HCV
- adenoviruses • rotaviruses • papovaviruses • polioviruses

Contact times

Effective against	Contact times
Bacteria, fungi (in accordance with DGHM Guidelines with short contact times)	1 min
MRSA Multiresistant Staphylococcus aureus strains	1 min
TB	1 min
Fully virucidal*	30 min
Effective against enveloped viruses* (incl. HIV, HBV, HCV)	30 sec
Adenoviruses	2 min
BVDV	30 sec
Rotaviruses	30 sec
Polioviruses	30 min
Polyoma SV 40	15 min
Vaccina viruses	30 sec
DGHM list / certificate Surface disinfection in hospitals and medical practices	5 min
ÖGHMP Expertise No. 95/8	1 min

*) In accordance with the Robert Koch Institute guideline – German Federal Health Gazette 01/2004

mikrozid® AF jumbo wipes

Product data

Composition: One dispenser or refill pack contains 200 ready to use wipes.
1 g of solution contains: 0.25 g Ethanol (94 %), 0.35 g 1-Propanol

Chemical / physical data:

Appearance of solution:	clear, colourless liquid
Density (20 °C):	approx. 0.895
Viscosity:	< 15 sec. (run-out time acc. to DIN 53 211)
Thermostability:	- 5 °C to + 40 °C
pH:	approx. 6.0
Flash point:	27 °C (DIN 51 755)

Instructions for use

Wipe areas thoroughly with the impregnated cloth; ensure thorough wetting and leave to work. If necessary, use several wipes large areas.

Labelling according to EC Directives

R10: Flammable

Special advice

Avoid contact with the eyes. Store out of the reach of children.
Do not use on alcohol-sensitive surfaces (e. g. acrylic glass).


Environmental information

Schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining our high quality standards.

Expert opinions and information

You can find an overview of all the expert opinions/literature to do with Mikrozyd® AF jumbo wipes online at www.schuelke.com

For any queries you might have about the product or using it please call +49 40 / 521 00-666

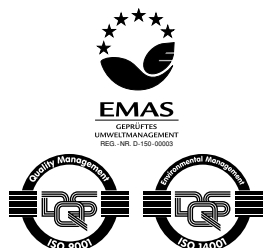

Schülke & Mayr GmbH is certified according to DIN EN ISO 9001, DIN EN ISO 14001 and DIN EN ISO 13485 (Reg.-No. 004567-MP23) and has a validated environmental system in accordance with the Eco Audit Regulation (Reg.-No. DE-150-00003).

Schülke & Mayr GmbH
22840 Norderstedt, Germany
Phone (+49) 40 - 521 00-0
Fax (+49) 40 - 521 00-318
www.schuelke.com
mail@schuelke-mayr.com

Schülke & Mayr UK Ltd.
1 Jenkin Road
GB-Sheffield S9 1AT
Phone (+44) 1142 - 54 35-00
Fax (+44) 1142 - 54 35-01
mail.uk@schuelke-mayr.com

Schülke & Mayr Ges.m.b.H
Seidengasse 9
1070 Wien, Austria
Phone (+43) 1 - 523 25 01-0
Fax (+43) 1 - 523 25 01-60
office@schuelke-mayr.at